

Clinician Quick Reference

BodyGuard™ 323
Ambulatory Infusion Pump


Loading the Administration Set



1. Open pump tubing door by pulling down on door latch located on right side of pump.



2. When loading set, reservoir bag will be on left side of pump.

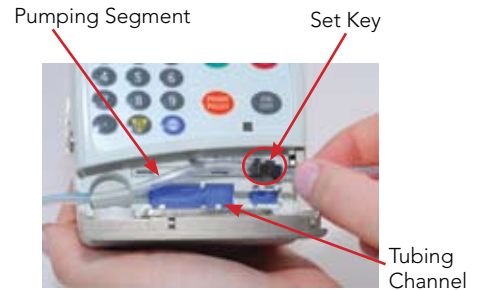

3. Locate pumping segment of set. It consists of softer segment of tubing about 4 ½ inches long. At right end of segment is black set key.



4. Match set key to slot. Set key fits into slot of same shape located in pump tubing channel.



5. Push key fully into slot.

Important: During loading, keep segment straight but do not stretch it.



6. Press rest of segment into tubing channel.

7. Close pump tubing door until you hear it click shut.

Important: While closing door, keep segment straight but do not stretch it.

8. Prime set (see illustrated steps on reverse side of sheet):

- Make sure patient is not connected to administration set
- Turn pump power ON and press PRIME/PIGGY
- Press START/OK
- Enter priming volume using pump keypad
- Press START/OK to start priming
- Display screen shows priming progress
- Repeat these steps for additional priming

Clinician Quick Reference



Priming the Administration Set

Screen Display

1. Make sure that patient is not connected to administration set.

2. Turn pump power ON and press



3. Press



4. Use pump keypad to enter volume of fluid required to prime set.
For priming volume, check tubing package.

5. Press




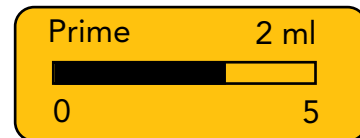
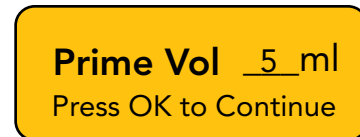
to start priming procedure.

Display screen shows priming progress.

6. For additional priming volume, repeat steps 1 – 5.

Note:

- To stop priming at any time, press
- 
- If line must be primed, it can only be primed on pump.
 - When pump is locked, patient can prime set in all operating modes.



For 24/7 clinical and technical support, call: 877.263.0111



CME America, LLC
14998 W. 6th Ave., Suite 830
Golden, CO 80401 USA



CAUTION: Before use, the user must become thoroughly familiar with the information contained in the device operating instructions. Note: This Clinician Quick Reference provides a convenient information summary. For complete information on proper use of the BodyGuard pump and administration set, refer to the device operating instructions supplied with the product. © 2008 by CME America, LLC. All rights reserved. The following are trademarks of CME America, LLC: BodyGuard, BodyComm, TSyringePump, the CME America logotype with "swirl" logomark and the tag line "Drug Delivery Designed for the Way You Deliver Care." **SM-0015 07/2008**